



LAUNDRY INSTRUCTIONS

How to wash your Sacci-products

Bags from SACCI can be machine washed at 40°C and 60°C. Do not use the spin cycle or tumble dry. Instead, allow the bag to hang dry.

NOTE! Use as large a washing machine as possible, preferably at a professional laundry service or use a machine larger than 7 kilos. For bags with plastic- or metal inserts, professional laundry service is recommended.

Specifically for emergency medical bags:

In case of blood or vomiting on or inside the bag, first rinse off with water, then scrub/wipe with Virkon or similar germicidal agent. Then wash the bag in a washing machine at 40°C or 60°C.

Remove the medicine holder and other loose partitions. Wash bag and medicine holder separately. The bag can be left open when washing.



From Torsång to the world. With our roots in between the forests and the winding river in Torsång, the family company has delivered backpacks and carrying equipment for extreme conditions for over a hundred years. Now as then, it's the craftsmanship, innovative thinking and quality of our products that make us renowned.

Sacci Ryggsäckar AB, Gamla Tunavägen 8, S-784 36 Borlänge, SWEDEN
Phone: +46 (0) 243-932 50, sacci@sacci.se

WWW.SACCI.SE

SACCI
SINCE 1914